



Bodygram+ enterprise



Bodygram+ enterprise

Akern Bodygram+ enterprise is a computer software for body composition analysis developed to satisfy the needs of an organization rather than individual users. Organizations usually hold strict web policies (web-based content access control and management) are used to share data among users placed in different workstations.

Comparison chart

Bodygram +	Bodygram + Enterprise
Bodygram Plus ClickOnce installs the software files in the User/Documents/BodygramPlus folder	Bodygram Plus enterprise Windows Installer installs the software files in c:/Program Files(x86)/Akern/BodygramPlus folder
The database, license, backups and settings are all placed in the same folder.	The database, license, backups and settings are placed in c:/ProgramData/Akern/BodygramPlus folder. ProgramData is a standard hidden folder used from the windows applications to share data between users.
Each ClickOnce application is installed in per-user, per-application cache folder. This implies that multiple users on a shared PC will need to install Bodygram PLUS on each user.	Windows installer package allows to install the software on ProgramFile(x86) folder. The IT staff can easily operate on sharing policy to allow PC users to access the software without administrator privileges and multiple installation
1 installation for each PC user	1 PC installation , irrespective from the number of users
Requires internet connection	doesn't require internet connection
The installation package can be downloaded from www.akern.com , (free internet connection required)	The installation package in a simple .exe file, that can be saved on multiple supports as usb pen, cd-rom etc.



Bodygram+ enterprise



The software is supplied along with the device on an USB pen .

The number of user-license available is 15.

Software registration: may be done via web (when an open internet connection is available on the pc) or offline.

Offline license files can be also created before installation by providing the customer account data ahead of the visit.

Customer account data to be provided are the following:
name - surname – institution – email -device model + device serials number – full address – country - telephone – mobile phone, and a password to login .

Upgrade policy: upgrades and updates are not automatic and may not be free of charge. A Maintenance Agreement is generally purchased on a yearly basis.

MINIMUM SYSTEM REQUIREMENTS



OS: Microsoft Windows VISTA, Windows 7, Windows 8, 8.1 PRO, Windows 10 – not compatible with Win 8RT.

Minimum screen resolution 1366 x 800 (recommended 1920 x 1080 HD)

Dual Core 1 GHz Processor

2 GB RAM (4 GB recommended)

1.5 GB free disk space

Requirements: Microsoft .NET framework 4, SQL engine.



MEDICAL DEVICE CLASSIFICATION

Bodygram Plus is classified as a CLASS I medical device (Directive 93/42/EEC);

Class A medical device under EN 62304:2006.